



Sleep Apnea patients helping one another

[Home](#)[Adjust CPAP Pressure](#)[Apnea Board Forum](#)[Apnea Board Wiki](#)[Sleep Apnea Info](#)

[Click here to enter the Apnea Board Forum](#)

We encourage you to [register](#) as a member of **Apnea Board** (it's totally free)

Instructions for entering the CPAP Setup / Clinician Menu of the Philips-Respironics PR System One Series Machines

(should work for most **PR System One** machines, such as the REMstar Auto, REMstar DS150, BiPAP ST, BiPAP AVAPS, BiPAP Auto with Bi-Flex, BiPAP Pro with Bi-Flex, REMstar Pro with C-Flex & REMstar Plus)

To get the **Clinician Setup Manual** for your CPAP Machine [CLICK HERE](#)

For instructions on how to **set the optimum pressure** on your CPAP Machine, [CLICK HERE](#)

Step 1:

On top of the CPAP machine, there is an LCD screen menu.
Select the appropriate quadrant by turning the knob. (see Picture 1):



Picture 1

Step 2:

Turn the knob to select "**setup**".

Press down the **knob** and the **arrow key** simultaneously until you hear the CPAP machine beep twice. (see Picture 2)



Picture 2

Step 3:

This will take you to a second menu.

Select "**setup**" again by turning the knob and then pressing the round knob. (see Picture 3)



Picture 3

Step 4:

This will take you to the CPAP clinician setup options. (see Picture 4)



Picture 4

Scroll down by turning the knob.

Make a selection by pressing the knob down, which will activate the option.

Change the setting by turning the knob again (*for instance, from a 5 minute to 10 minute ramp time*).

Press knob to lock that choice in.

Get CPAP help on the **Apnea Board Forum**: [CLICK HERE](#)

Review of the PR System One REMstar CPAP machines



PR System One REMstar Auto Series 60 Heated Hose



DISCLAIMER & LEGAL NOTICE: BY USING THESE CPAP SETUP DIRECTIONS AND SETUP GUIDES, YOU UNDERSTAND THAT YOU ARE PERSONALLY ASSUMING ALL RISK FOR USING INFORMATION ON THIS WEB PAGE AND IN THESE DOCUMENTS AND WILL NOT HOLD APNEA BOARD, ITS MEMBERS, OWNERS, ADMINISTRATORS AND MODERATORS LIABLE FOR ANY PROBLEMS, DAMAGES, PHYSICAL HARM, (INCLUDING DEATH), MENTAL HARM OR ANY OTHER HARM THAT MAY COME TO YOU, YOUR FAMILY MEMBERS, FRIENDS, CO-WORKERS OR ANY OTHER PERSONS WHO MAY RECEIVE THIS INFORMATION FROM YOU. YOU MAY NOT USE, COPY OR DOWNLOAD THESE DOCUMENTS UNLESS YOU HAVE A LEGAL RIGHT TO DO SO UNDER ALL APPLICABLE U.S. LAWS. INFORMATION CONTAINED ON THIS WEB PAGE, APNEA BOARD FORUMS AND ON THE APNEABOARD.COM WEB SITE SHOULD NOT BE CONSIDERED AS MEDICAL ADVICE. ALWAYS SEEK THE ADVICE OF A QUALIFIED PHYSICIAN BEFORE SEEKING TREATMENT FOR ANY MEDICAL CONDITION, INCLUDING SLEEP APNEA. ALWAYS SEEK THE ADVICE OF A QUALIFIED PHYSICIAN BEFORE CHANGING PRESSURE ON ANY MEDICAL DEVICE, INCLUDING CPAP MACHINES.

We need your help

If you can, please donate a couple of dollars to Apnea Board by clicking on the "Donate" button below:

DONATE to Apnea Board

[Go back to Apnea Board](#)